## **ENGLISH**

---

### **USES**

The Spectra Wrist Brace can be used to treat tendonitis, Carpal Tunnel Syndrome, arthritis, strains and sprains, post cast healing and other selected wrist injuries.

### **CAUTION**

Patient should be instructed to inspect their hand, wrist, forearm and this product before and after each use.

This product is designed to be worn directly on the skin. If any problems occur with this product, please contact your certified medical professional.

### **PHOTO CAPTIONS**

- 
- 
- A. Adjustable contact closure strap
  - B. Webspace contact closure strap
  - C. Stockinette
  - D. Aluminum palmer stay
  - E. Aluminum dorsal stay

### **FITTING INSTRUCTIONS**

1. Undo both the contact closures straps.
2. Place hand into the support and ensure stockinette is positioned on top of the hand.
3. Align the aluminum palmer stay to the palm of the hand. Adjustments on the aluminum palmer stay can be performed by a certified medical professional.
4. Fasten the adjustable contact closure strap and secure to a snug fit. Adjustments may be necessary as hand, wrist and thumb swelling patterns change.
5. Fasten webspace contact closure strap across aluminum

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EN- This product has been designed and tested based on single patient usage. It is a single-use device and should never be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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dorsal stay. Adjust for comfort.

## **WASHING INSTRUCTIONS**

1. This product may be cleaned using a mild detergent and cold water. Aluminum palmer stay should be removed before washing. Do not bleach or use strong detergent. Do not wash in washing machine.
2. Air dry only. Do not squeeze or wring out the brace. Do not place in a dryer or near a heater.

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