# Stomacare

EasyOpener *Plus*

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**EN** Instructions for use

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**NO** Bruksanvisning

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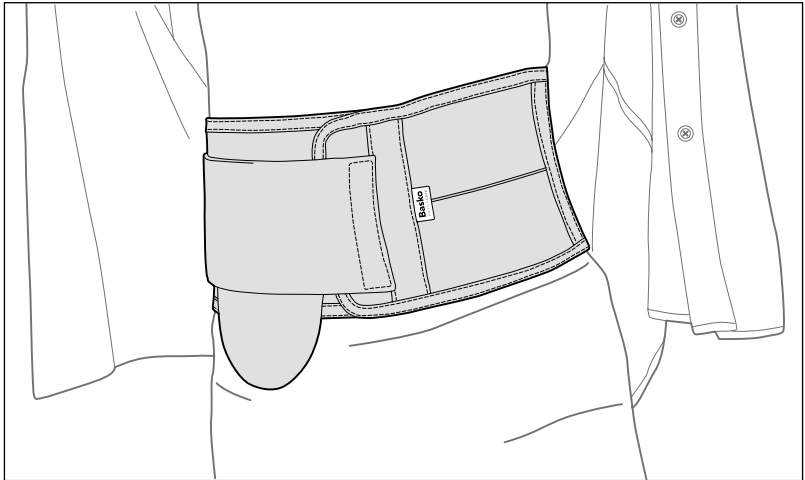
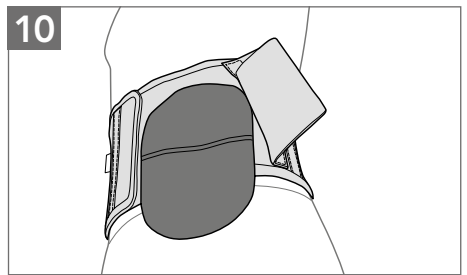
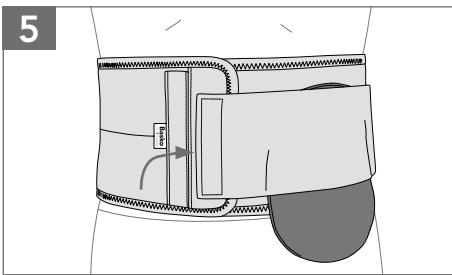
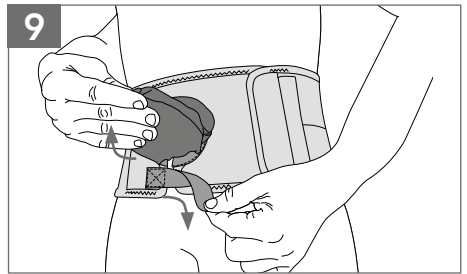
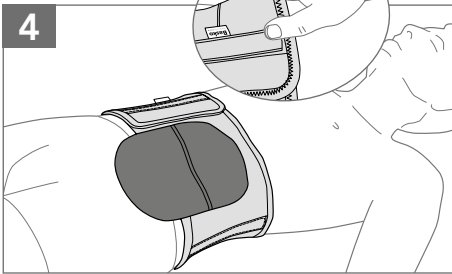
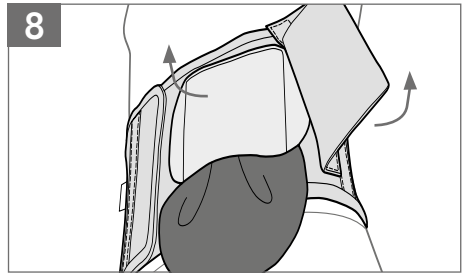
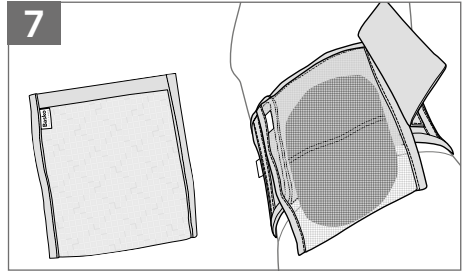
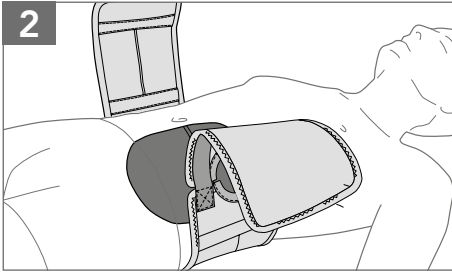
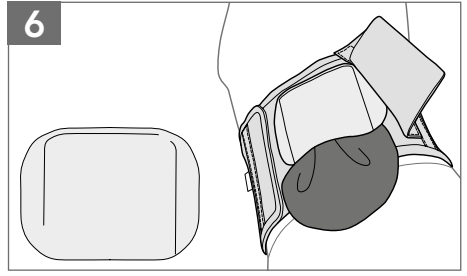
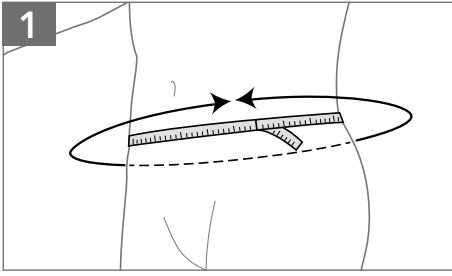
**Basko**  
Healthcare

[basko.com](http://basko.com)

## Overview Stomacare EasyOpener Plus

Height	Ø Aperture	Size	Item number	Side	Item number	Side
15 cm / 5.9"	60 mm / 2.4"	Small	S3156004L-S	Left	S3156004R-S	Right
15 cm / 5.9"	60 mm / 2.4"	Medium	S3156004L-M	Left	S3156004R-M	Right
15 cm / 5.9"	60 mm / 2.4"	Large	S3156004L-L	Left	S3156004R-L	Right
15 cm / 5.9"	60 mm / 2.4"	X-Large	S3156004L-XL	Left	S3156004R-XL	Right
15 cm / 5.9"	60 mm / 2.4"	XX-Large	S3156004L-XXL	Left	S3156004R-XXL	Right
15 cm / 5.9"	60 mm / 2.4"	XXX-Large	S3156004L-XXXL	Left	S3156004R-XXXL	Right
15 cm / 5.9"	60 mm / 2.4"	XXXX-Large	S3156004L-XXXXL	Left	S3156004R-XXXXL	Right
15 cm / 5.9"	72 mm / 2.8"	Small	S3157204L-S	Left	S3157204R-S	Right
15 cm / 5.9"	72 mm / 2.8"	Medium	S3157204L-M	Left	S3157204R-M	Right
15 cm / 5.9"	72 mm / 2.8"	Large	S3157204L-L	Left	S3157204R-L	Right
15 cm / 5.9"	72 mm / 2.8"	X-Large	S3157204L-XL	Left	S3157204R-XL	Right
15 cm / 5.9"	72 mm / 2.8"	XX-Large	S3157204L-XXL	Left	S3157204R-XXL	Right
15 cm / 5.9"	72 mm / 2.8"	XXX-Large	S3157204L-XXXL	Left	S3157204R-XXXL	Right
15 cm / 5.9"	72 mm / 2.8"	XXXX-Large	S3157204L-XXXXL	Left	S3157204R-XXXXL	Right
15 cm / 5.9"	85 mm / 3,3"	Small	S3158504L-S	Left	S3158504R-S	Right
15 cm / 5.9"	85 mm / 3,3"	Medium	S3158504L-M	Left	S3158504R-M	Right
15 cm / 5.9"	85 mm / 3,3"	Large	S3158504L-L	Left	S3158504R-L	Right
15 cm / 5.9"	85 mm / 3,3"	X-Large	S3158504L-XL	Left	S3158504R-XL	Right
15 cm / 5.9"	85 mm / 3,3"	XX-Large	S3158504L-XXL	Left	S3158504R-XXL	Right
15 cm / 5.9"	85 mm / 3,3"	XXX-Large	S3158504L-XXXL	Left	S3158504R-XXXL	Right
15 cm / 5.9"	85 mm / 3,3"	XXXX-Large	S3158504L-XXXXL	Left	S3158504R-XXXXL	Right

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# EasyOpener *Plus* Stomacare Ostomy Belt

## Instructions for use

Your Stoma, wound nurse or physician has advised you to start wearing a stoma belt. For optimal care, the user should read these instructions carefully before applying the belt. This product is a medical device and is intended for use by one person only.

## Intended use of the product

The EasyOpener *Plus* belt is mainly used postoperatively to support the abdominal wall and prevent the formation of hernia and prolapse, for example during physical activities where pressure on the abdominal wall is higher (sports, gardening, lifting, vacuuming, etc.). The EasyOpener *Plus* belt offers the wearer a high level of wearing comfort, a safe and secure feeling in daily life.

## Packaging content

First check that the contents of the packaging are complete and undamaged. If this is not the case, please contact the person from whom you purchased the product directly.

The packaging contains the following parts:

- Stomacare EasyOpener *Plus* Ostomy Belt
- Instructions for use

## Applying belt

To wear the EasyOpener *Plus* belt correctly, you should preferably apply the belt while lying down. In case of a fracture, in many cases it will sink back into the abdomen and the belt provides counter pressure. This way the fracture is optimally supported. The belt is closed in such a way that you can still slide your hand in between. The belt is then sufficiently tight to provide support but still wears comfortably.

### Fig. 1

The circumference of the abdomen must be measured to select the correct size. Use the size chart to determine which size is the best fit.

### Fig. 2

Attach the ostomy bag against the abdominal wall and check that it is comfortable and properly adhered. Place the ostomy belt on the bed (or put it on loosely) and lie down on top of it.

### Fig. 3

Fold the tip of the stoma pouch into a point and guide it from the inside of the belt out through the stoma aperture, with flange remaining at inside of the belt.

### Fig. 4

Make sure the stoma pouch is nice flat and without folds. Next, pull the front Velcro closure over the abdomen and close it at the desired tension. Check whether it's still possible to insert a hand between the belt and your abdomen. If necessary, adjust the tension. After completion you can stand up.

### Fig. 5

Now place the end of the prolapse cover with its Velcro on the velour plate. Stretch the prolapse cover to the desired tension over the pouch and secure the Velcro fastener.

### Fig. 6 (optional)

If chosen, first position the Protector (accessory that protects you from unwanted pressure on the stoma, e.g. due to seat belt) over the stoma pouch and press it tightly with the Velcro onto the belt. Then reattach the prolapse cover as described.

### Fig. 7 (optional)

You can also choose to apply a BellyBag (an accessory that helps to 'carry' the pouch, providing extra comfort and support) to the velour panel. Then reattach the prolapse cover as described.

## Changing the ostomy pouch

### Fig. 8

Open the prolapse cover and secure it to the velour panel next to the ostomy aperture with the Velcro hook. If applicable, remove the Protector or BellyBag from the bandage.

### Fig. 9

Lift the ostomy pouch slightly forward and upward and open the special Velcro closure right underneath the stoma aperture.

### Indication

- Abdominal wall weakness
- Cicatricial and Stoma hernia
- Colon Conduit
- Colostomy
- Enterostomy
- Ileam Conduit
- Ileostomy
- Prolapse
- Ureterocutaneostoma
- Urostomy

### Contraindications

There are no contraindications known.

### Additional product information / area of use

The belt is not suitable for use in salt water.

### Existing user risks

No user risks are known. However, the user must regularly check if pressure points or other skin irritations occur and must immediately contact the oEasyOpener, wound nurse or technician if so.

### Warning

The belt must not be worn too tight and must be tensioned in such a way that the hand can still be inserted. The bandage should not come into direct contact with open wounds and should be worn on dry skin. The use of lotion or creams is not recommended and should first be discussed with your stoma, wound nurse or physician.

### Maintenance advice

If the belt is not worn, it is important to store it with the closures closed. To elongate the life of the belt, it is important that the Velcro straps remain free of dirt, dust etc. to continue to guarantee good adhesion.

### Materials used

Latex-free; Coolmax (polyester, polyamide, elastane); Velour (polyester); Plastic (POM, TPE).

### Washing instructions

We recommend washing the belt in warm water at 40° C with a mild detergent. To prevent damage to the fabric, the Velcro closures must be closed prior to washing.

### Fig. 10

Pull the adhesive edge of the ostomy pouch away from the skin and remove the pouch along the bottomsides of the belt. To apply a new ostomy pouch, follow the instructions as described and close the Velcro closure under the ostomy aperture. Then reattach the prolapse cover as described.

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